

# Mepilex® Border Ag



## THE EFFECTIVE ANTIMICROBIAL BORDERED FOAM DRESSING WITH SAFETAC® TECHNOLOGY

- Minimizes pain and trauma at dressing changes<sup>1,2,3</sup>
- Rapid and sustained antimicrobial activity<sup>5</sup>
- Effective exudate management

### Safetac® technology layer

- Reduces pain and trauma during dressing changes<sup>1</sup>
- Does not adhere to the moist wound bed, only to dry skin<sup>3</sup>
- Seals the wound margins and reduces risk of maceration<sup>4</sup>

### Polyurethane backing film


- Moisture vapor permeable
- Moisture proof
- Bacteria and viral barrier<sup>†</sup>

### Multi-layered absorbent foam pad

- The compressed foam with silver absorbs and spreads the exudate while allowing silver to be released
- The super absorbant layer wicks and retains exudate
- The nonwoven layer spreads and retains fluid

### Powerful silver performance

- Antimicrobial activity within 30 minutes and sustained effect for up to 7 days<sup>5</sup>

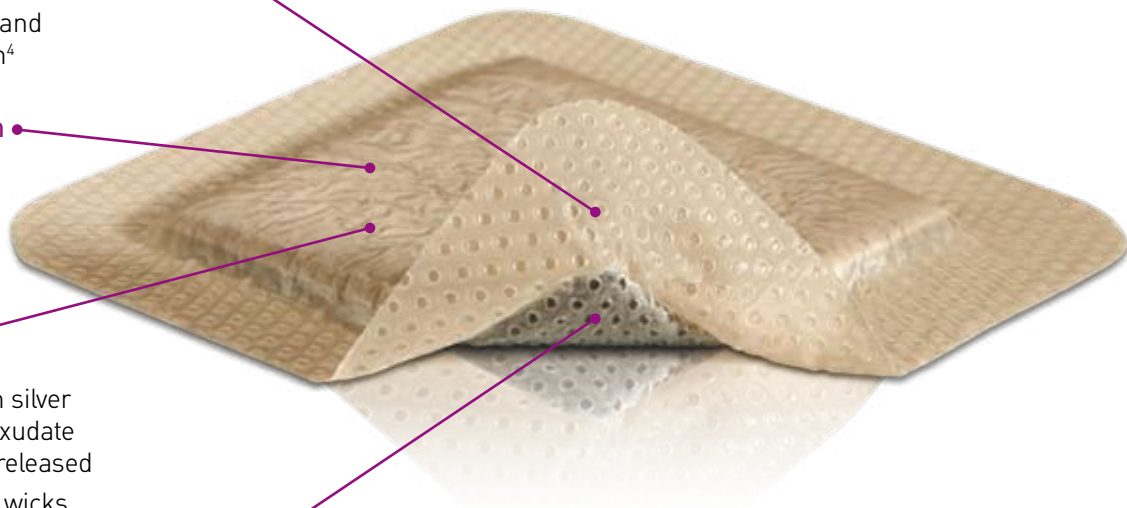

**Safetac®**  
TECHNOLOGY

Starts to work in **30** minutes Sustained release for **7** days

### Safetac® technology

#### Safetac® technology. Less pain and less trauma.

Safetac® is a patented adhesive technology that minimizes pain to patients and trauma to wounds. Safetac® technology is available exclusively on Mölnlycke Health Care dressings, including Mepilex® wound care dressings, Mepitel®, Mepiform® and Mepitac®.

For more information visit [www.molnlycke.us](http://www.molnlycke.us)




Painful skin stripping with traditional adhesive




No skin stripping with Safetac® technology

## How to use Mepilex® Border Ag

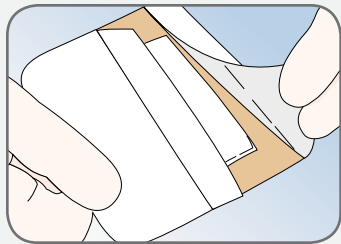

Clean the wound area. Skin barrier products not necessary. Remove the release film.

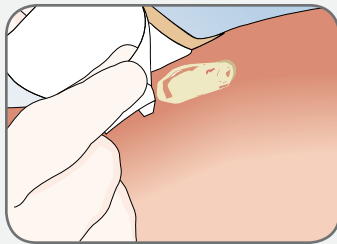

Apply the adherent side to the wound. Do not stretch.



## How Mepilex® Border Ag works

**Mepilex® Border Ag** is an antimicrobial all-in-one foam dressing that effectively absorbs and retains exudate and maintains a moist wound environment. The **Safetac®** layer seals the wound edges, which prevents the exudate from leaking onto the surrounding skin and minimizes the risk of maceration. The **Safetac®** layer ensures that the dressing can be changed without damaging the wound or surrounding skin or exposing the patient to additional pain.

**Mepilex® Border Ag** inactivates wound related pathogens within 30 minutes with sustained effect up to 7 days.

By reducing the number of microorganisms,

**Mepilex® Border Ag** may also reduce odor.

## Benefits of Mepilex® Border Ag

- Minimizes pain and trauma at dressing changes
- Effective, sustained release of silver
- Self-adherent - no secondary fixation required
- Shower proof
- May be used under compression bandages
- Promotes patient comfort during wear
- May remain in place for several days depending on the condition of the wound
- May be lifted and adjusted without losing its adherent properties
- Low potential for skin irritation and allergy

## Areas of use

**Mepilex® Border Ag** is designed for medium to high exuding wounds such as leg and foot ulcers, pressure ulcers, partial thickness burns, traumatic and surgical wounds, where a moist environment, exudate management, gentle fixation and antimicrobial action is indicated.

† Microbes larger than 25 nm

‡ Packaged sterile in single packs

**Safetac®**  
TECHNOLOGY

5550 Peachtree Parkway, Suite 500, Norcross, GA 30092 | 800.882.4582 | [www.molnlycke.us](http://www.molnlycke.us) | [www.mhcwoundcare.com](http://www.mhcwoundcare.com)

The Mölnlycke Health Care, Mepilex, Mepitel, Mepiform, Mepitac, and Safetac, names and respective logos are registered trademarks of Mölnlycke Health Care AB. Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. © 2010. Mölnlycke Health Care AB. 1.800.882.4582.

## Mepilex® Border Ag Ordering Information†

Product Code	Size	Pieces/box	Pieces/case	HCPCS
395290	3" x 3" (7.5 x 7.5 cm)	5	70	A6212
395390	4" x 4" (10 x 10 cm)	5	50	A6212
395490	6" x 6" (15 x 15 cm)	5	50	A6213
395690	6" x 8" (15 x 20 cm)	5	45	A6213
395890	4" x 8" (10 x 20 cm)	5	35	A6212
395790	4" x 10" (10 x 25 cm)	5	35	A6212
395990	4" x 12" (10 x 30 cm)	5	25	A6213

## Mepilex® Border Sacrum Ag

382090	7.2" x 7.2" (18 x 18 cm)	5	40	A6213
382490	9.2" x 9.2" (23 x 23 cm)	5	25	A6213

## Precautions

- Do not use on patients with a known sensitivity to silver.
- Do not use **Mepilex® Border Ag** during radiation treatment or examinations e.g., X-ray, ultrasound, diathermy or Magnetic Resonance Imaging.
- Do not use **Mepilex® Border Ag** together with oxidizing agents such as hypochlorite solutions or hydrogen peroxide.

## References:

1. White R., A Multinational survey of the assessment of pain when removing dressings. *Wounds UK* 2008; Vol 4, No 1.
2. Dykes PJ et al. Effects of adhesive dressings on the stratum corneum of the skin. *J Wound Care* 2001; 10(2):7-10.
3. Dykes PJ. The effect of adhesive dressing edges on cutaneous irritancy and skin barrier function. *J Wound Care* 2007; 16(3):97-100.
4. Meaume S et al. A study to compare a new self adherent soft silicone dressing with a self adherent polymer Management, 2003.
5. External lab report: NAMS A 09C 29253 01/09C 29253 02.

 **MÖLNLYCKE®**  
HEALTH CARE

US038611010 - 270320 11.10